


# Place your dosimeter badge properly

It is important to wear your RaySafe i3 real-time dosimeter properly to obtain accurate radiation readings. Follow these recommendations to ensure you're wearing your badge correctly.

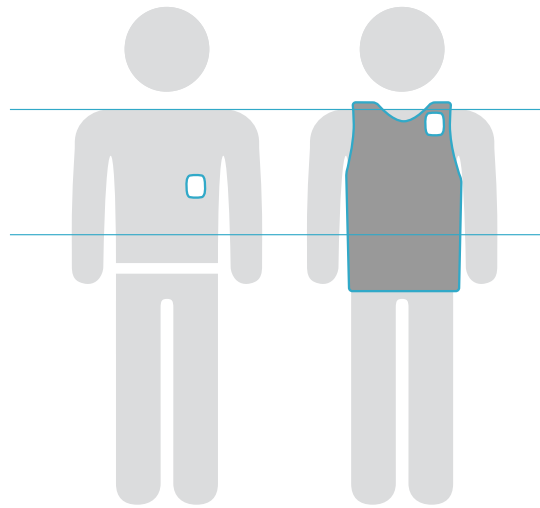

Wear dosimeter between **shoulders** and **waist**.

If using a lead apron and/or thyroid collar, wear **outside** protection garments

## RAYSAFE I3 REAL-TIME DOSIMETRY SYSTEM

Monitor your X-ray dose exposure instantly.

The RaySafe i3 is an active dosimetry system that provides real-time insights about radiation exposure, helping medical staff and physicians evaluate and directly adjust their behaviors and more effectively use all the radiation reduction solutions provided in the room.

Learn more at: [RaySafe.com](https://www.raysafe.com)

