

Technical Data

RaySafe Pro-Mam Biopsy Phantom

This phantom was designed to provide a fast and easy way to test image quality on digital biopsy mammography units without having to do multiple exposures.

Key features


- The fast and easy way to test image quality on digital biopsy mammography units
- Designed to meet ACR requirements for use in the Stereotactic Breast Biopsy Accreditation Program, but does not hold an accreditation
- The phantom contains test objects that are similar to those found in the Mammographic Accreditation Phantom specified by ACR
- The extended top edge of the phantom allows for easy positioning on recumbent biopsy units
- Can be used in upright and prone machines
- The phantom's small size allows it to be imaged in its entirety in a single exposure when used with a digital biopsy unit – one exposure is all you need



Detect imaging changes

The small-sized RaySafe Pro Mam Biopsy phantom permits fast and easy evaluation of conventional and digital image quality. Image quality can be evaluated with a single exposure.

Test objects comply with requirements specified in the ACR 1999 Stereotactic Breast Biopsy Quality Control Manual, the Mammographic Accreditation Phantom specified by the ACR. Regular quality control performed with the RaySafe Pro Mam Biopsy phantom allows you to detect any changes in imaging quality, enabling you to maintain the system so it remains at peak performance.

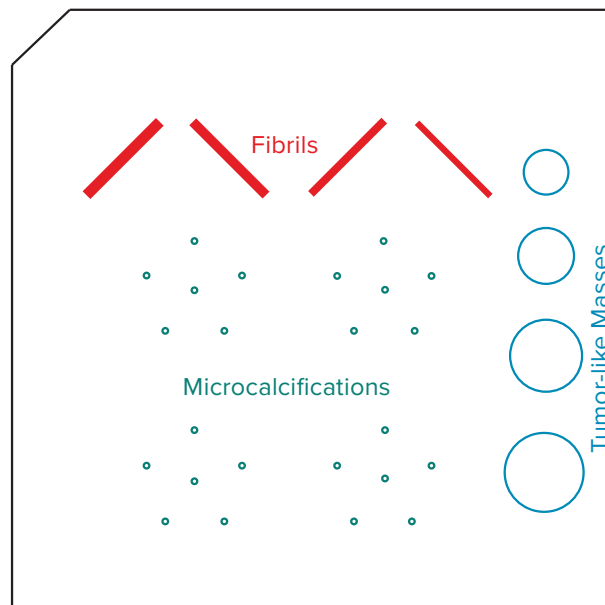

Specifications

Phantom body	
Material	Acrylic (PMMA)
Dimensions and weight	
Overall dimension (h x w x d)	46 x 80 x 70 mm (3.1 x 2.8 x 1.8 in.)
Cut-out dimension (h x w x d)	6.25 mm x 60 x 60 (0.25 x 2.4 x 2.4 in.)
Simulates	42 mm (1.7 in.) compressed breast of average glandular/adipose composition
Weight	316 g (0.70 lb)
Wax insert test elements	
Nylon fiber diameters (4)	0.93, 0.74, 0.54, 0.32 mm (0.037, 0.029, 0.021, 0.013 in.)
Aluminum oxide microcalcifications specks (4)	0.54, 0.32, 0.24, 0.20 mm (0.021, 0.013, 0.0094, 0.0079 in.)
Tumor like masses (thickness) (4)	0.25, 0.50, 0.75, 1.00 mm (0.0098, 0.020, 0.030, 0.039 in.)
Standards	
Compliance	Complies with, but does not hold an accreditation: <ul style="list-style-type: none"> • ACR Mammography Accreditation Program Requirements • ACR Stereotactic Breast Biopsy Accreditation Program

Note: The phantom is made of a 6.25 mm (0.25 in.) wax block insert containing 12 sets of test objects, a 36.75 mm (1.45 in.) thick acrylic base, and a 3 mm (0.12 in.) thick cover.

The phantom approximates a compressed breast of average glandular/adipose composition.

Included in the wax insert are four aluminum-oxide (Al_2O_3) specks that simulate microcalcifications. Four different nylon fibers simulate fibrous structures and four different size lens-shaped masses simulate tumors.

Each phantom includes a 4 mm (0.16 in.) thick, Ø 10 mm (0.39 in.), 6 g (0.2 oz) acrylic contrast test disk, and a magnifying glass.

Ordering information

Model 18-250-01 RaySafe Pro-Mam Biopsy Phantom.

Includes a rotating support plate, and a comfortable carrying case.

RaySafe

We empower our everyday heroes to focus only on protecting lives.

Unfors RaySafe AB
Uggledalsvägen 29
427 40 Billdal, Sweden

For more information, contact us at:

+46 31 719 97 00
customerservice.se@raysafe.com
raysafe.com

©2023 RaySafe
Specifications subject to change without notice.
7/2023 22153b-en

Modification of this document is not permitted without written permission from Fluke Health Solutions.

