





Select RaySafe i3 On Screen Display View Options



If you have a RaySafe i3 OSD system delivered after mid-March 2025, it has software supporting multiple screen viewing options.

If you have an older device you need to download a zip file that includes the updated software package (xhub2_firmware_1.2.70.tar) along with the installation instructions (RaySafe_i3_hub_upgrade.pdf) and this document (Instruction select view OSD.pdf) from RaySafe.com using this [link](#):

Viewing options are:

- Session accumulated dose as a number (new)
- Dose rate history bars
- Both
- None

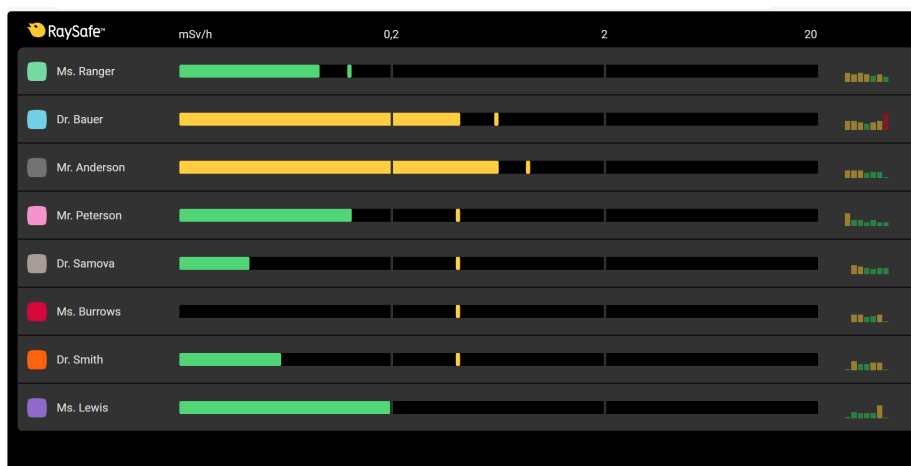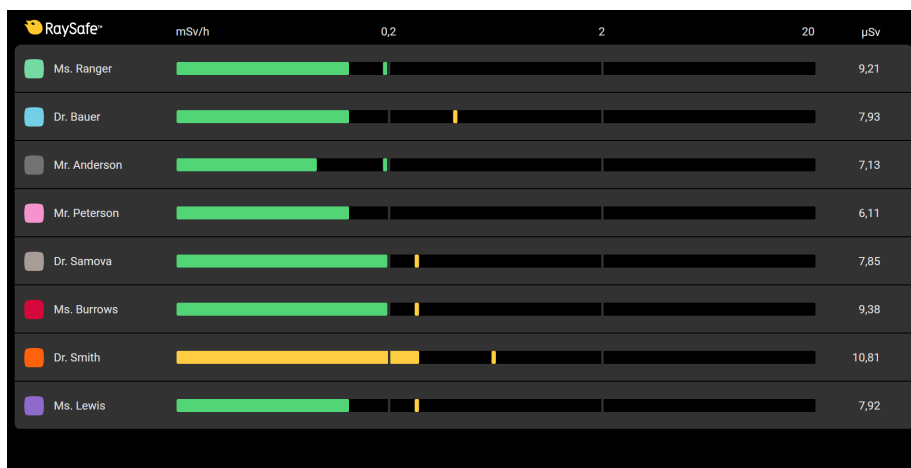

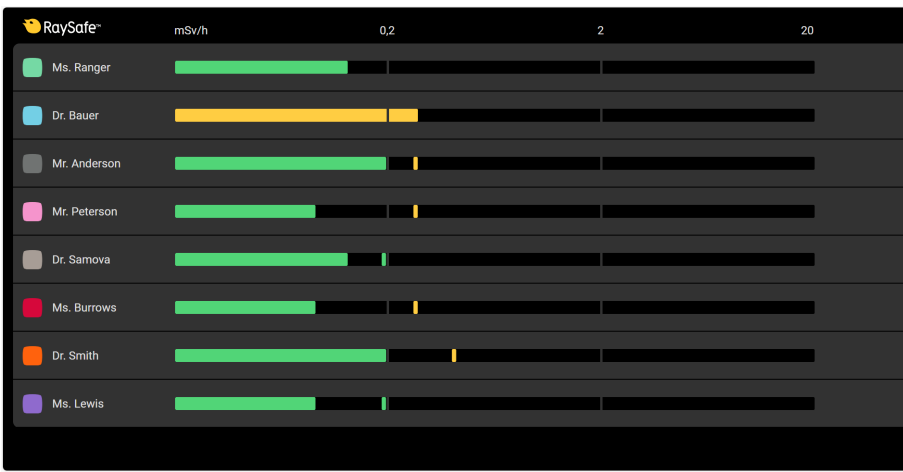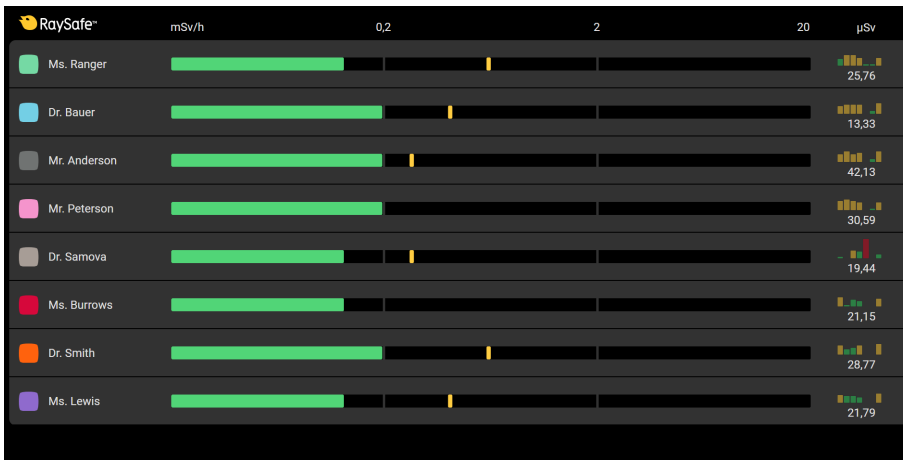

Once installed, you can select your preferred display option under Configuration > Real-Time View Options (see the screenshot below for reference). If you have any questions or require assistance, please do not hesitate to contact us.

RaySafe™

Sign out Save and reboot Auto logout: 55:55

Configuration Set-up Data export Log

Reference dosimeter

Serial number

Dosimeter communication

Radio range ☐ Nearby ☒ In lab ☐ Maximum range

Regional settings

Unit ☒ mSv ☐ rem

Language English

Decimal separator ☒ Point (.) ☐ Comma (,)

Real-time View access

Browser access ☒ Any IP address ☐ Only specified IP addresses

Note: Max 3 simultaneous connections recommended, Video Unit included.

Real-time View options

Procedure accumulated dose ☐ Show ☒ Hide

History dose rate bars ☒ Show ☐ Hide

Note: History bars are only available with a reference dosimeter configured.

Clear history

Clear all dose history and dosimeter information from the hub.

Note: No hub system settings will be affected.

Clear history and reboot

Service mode

Display dosimeter radio signal strength.

Activate for 20 minutes