


How to upgrade the RaySafe i3 Real-time Hub


Follow the steps below to upgrade the software in the RaySafe i3 Real-time Hub.

Tip! Visit www.raysafe.com for manuals and support contacts.


1. Log in to "System configuration" on the RaySafe i3 Real-time Hub.

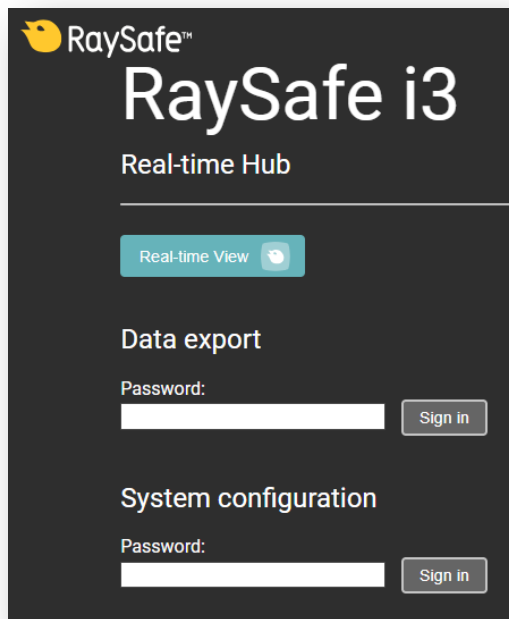

2. Click "Set-up" tab.

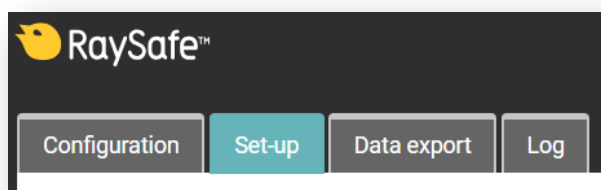

3. At the bottom of the page, click "Choose File" and select the update file ("xhub2_firmware_x.x.xx.tar") from your file system.
Click "Upload and reboot".

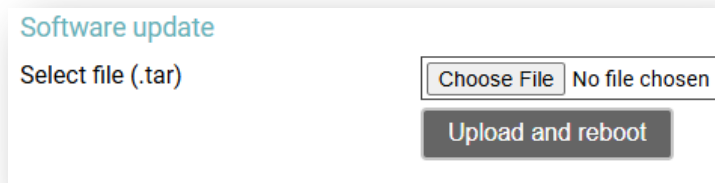A screenshot of a web interface for software updates. It features a title "Software update" in blue. Below it is a text input field labeled "Select file (.tar)". To the right of the input field are two buttons: "Choose File" and "No file chosen". Below these buttons is a large, dark grey button labeled "Upload and reboot".

Software update

Select file (.tar)

Choose File No file chosen

Upload and reboot

4. The update will take up to 10 minutes. After 10 minutes, reload the page in your web browser. Verify the new software version at the bottom of the page.